


HOW TO SCAN FOR A MICROCHIP

FOLLOW THESE STEPS FOR SCANNING A PET!


FIRST CHECK BETWEEN THE SHOULDERS:

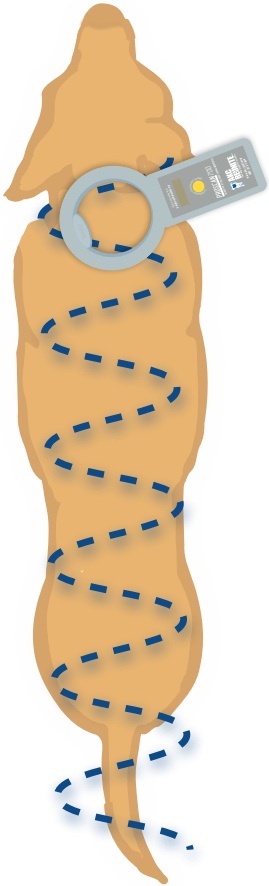

1. Using a Universal Scanner, start scanning slowly between the pet's shoulder blades, going side-to-side in an **"S" shaped pattern**.
2. If no chip is detected, **rotate the scanner 90 degrees** and scan again between the pet's shoulder blades.
3. Then, start at the pet's head, and slowly scan side-to-side all the way to the pet's tail, **making several passes** over each area.
4. When you find a microchip, always check where it is enrolled on petmicrochiplookup.com!

IF NO CHIP IS DETECTED:

1. Make sure you have **fresh batteries!** (We recommend Duracell® or Energizer®)
2. **Slowly scan** down the sides of the pet's leg, across its neck, along and under its chest and behind its front legs.

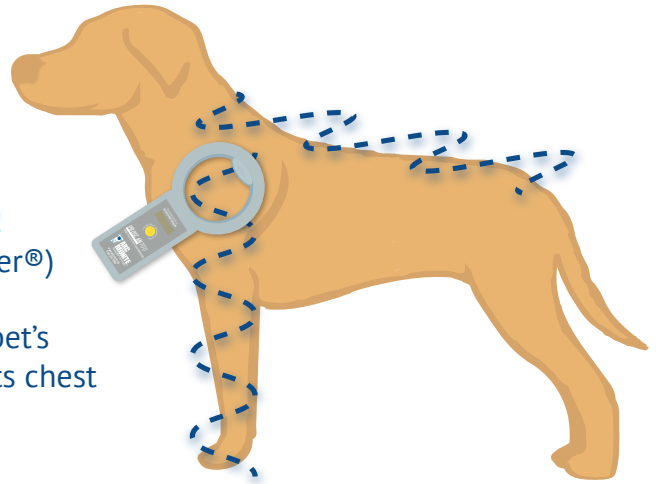

It can take up to **2 MINUTES** to detect a pet's microchip. **TAKE YOUR TIME!**



Always check the chip at petmicrochiplookup.org.
Brought to you by AAHA



PROSCAN 700
optimal read areas are the right and left sides of the circle.

QUICKSCAN 650
optimal read area is the red sensor on the scanner.

