


HOW TO SCAN FOR A MICROCHIP


FIRST CHECK BETWEEN THE SHOULDERS:

1. Using a Universal Scanner, start scanning slowly between the pet's shoulder blades, going side-to-side in an "S" shaped pattern.
2. If no chip is detected, **rotate the scanner 90 degrees** and scan again between the pet's shoulder blades.
3. Then, start at the pet's head, and slowly scan side-to-side all the way to the pet's tail, **making several passes** over each area.
4. When you find a microchip, always check where it is enrolled on petmicrochiplookup.com!

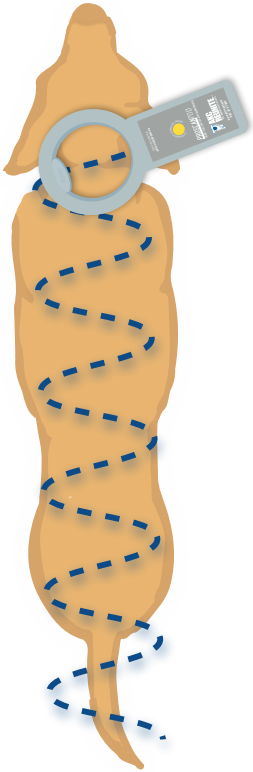

IF NO CHIP IS DETECTED:

1. Make sure you have **fresh batteries!** (We recommend Duracell® or Energizer®)
2. **Slowly scan** down the sides and behind the pet's legs, across its neck, and under its chest.

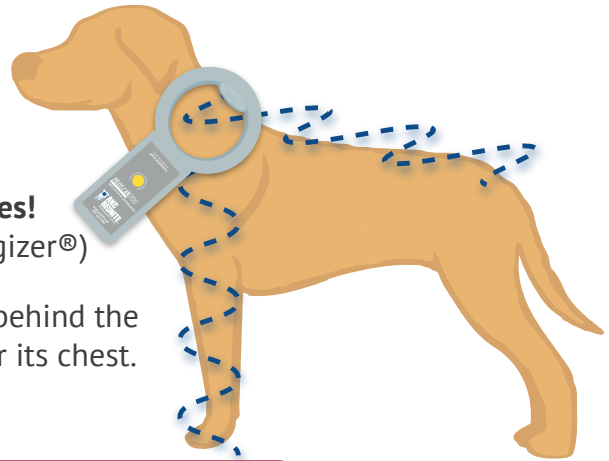

 = Antenna Location That Reads Microchip

PROSCAN 700




QUICKSCAN 650



WANDSCAN 900 REACH



WANDSCAN 900 PRO

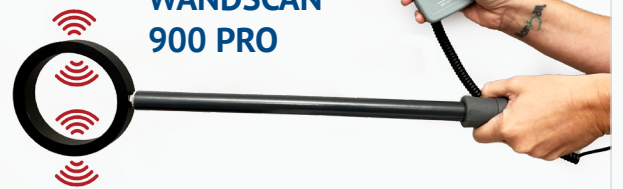

It can take up to **2 minutes** to detect a pet's microchip



Always check the chip at
aaha.org/petmicrochiplookup

Brought to you by AAHA



www.akcreunite.org
800-252-7894

FIND US ON:   